ACTIVE

Erection System^{NT®}

Medical Vacuum Erection Aid
Instructions for Use

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MEDintim

personal healthcare

Causes of Erectile Dysfunction and Impotence

Metabolic diseases: Diabetes mellitus, lipid metabolism disorders, thyroid disease, liver cirrhosis, renal failure.

Cardiovascular conditions: Coronary heart disease, high blood pressure (hypertension), arteriosclerosis, myocardial infarction (heart attack).

Neurological disorders: Stroke, Parkinson's disease, multiple sclerosis (MS), disc herniation, spinal cord injury, nerve damage caused by surgery (prostatectomy, partial removal of the rectum, cystectomy).

Psychological disorders: E.g. depression.

Urogenital malformations and diseases: Phimosis, hypospadias, neoplastic disease, rupture of the corpora cavernosa (penis fracture).

Surgical procedures in the abdominal, pelvic and urogenital area: Radical prostatectomy, vascular surgery in the abdominal cavity, interventions on prostate or testicles, bladder and bowel surgery.

Alterations of the corpus cavernosum: Peyronie's disease, induratio penis plastica (IPP)

Side effects of drugs: Beta blockers, ACE inhibitors, antidepressants, sedatives (tranquilizers), neuroleptics (antipsychotics) and others; smoking, alcohol abuse.

Product Information

The **ACTIVE Erection System^{NT}** is a medical device. It is used in erectile dysfunction in order to obtain an erection. The vacuum pump creates a negative pressure in the cylinder around the penis. This causes the corpora cavernosa to fill with blood, leading to artificial stiffening of the penis. Now one of the constriction rings is slid over the penis shaft. The

constriction ring prevents the blood from flowing back after removal of the cylinder, thereby maintaining the erection. With the erection thus maintained, sex and normal and satisfying sexual intercourse is possible. After sex, remove the constriction ring again. All parts are stored for re-use in the bag included in the delivery.

The **ACTIVE Erection System^{NT}** consists of the following parts:

- Cone
- Ring loader 1
- Ring loader 2
- Transparent cylinder
- Pump head, electric, with 3 batteries AAA 1.5 volts
- 4 constriction rings of different sizes (10, 12.5, 15 and 20 mm)
- Lubricant (e.g. SYLK)
- Instructions for use
- Storage bag
- DVD with animated film (Instructions for Use)

Application

The **ACTIVE Erection System^{NT}** is easy to use, but should initially be tried out in peace and alone. If both partners are familiar with the **ACTIVE Erection System^{NT}** later, there is a good chance for both to experience satisfying sexuality – even at an advanced age, in sickness or in case of physical disability.

Choosing the right constriction ring is important to maintain the best possible erection. If the selected ring is too large, the stiffening will be maintained only insufficiently. The erection declines or disappears

completely after slipping on the ring. If the ring is too tight, you may experience pain. In this case you should slide off the constriction ring and try again with a larger ring after the penis has stiffened again in the cylinder. The **ring loader 1** is somewhat narrower, the **ring loader 2** a little wider. The two ring loaders are suitable for all sizes of constriction rings. If both ring loaders are too narrow for the erect penis, you can slip the constriction ring directly onto the transparent cylinder.

The **cone** is only intended for slipping the rings onto the ring loaders. You can also slip the constriction ring directly by hand (without cone) onto the ring loaders.

Functional Elements

The motor for the vacuum pump is located in the pump head. Remove the pump head cover with a rotary movement, lift up the flap with the nameplate and insert the three 1.5-V batteries (AAA). Pay attention to the correct position (plus and minus poles). Make sure the poles match the symbols on the battery compartment.

Controls

Plus (+) and minus (-) buttons


There are (+) "on" and (-) "off" buttons on the pump head. Press and hold the (+) "on" button until the pump starts. Repeated pressing increases the suction force and therefore the vacuum in the cylinder. Repeated pressing of the (-) "off" button reduces the suction force and thus the vacuum. To turn the unit off, press and hold the (-) "off" button for 3 seconds. When the motor is switched on, two LEDs illuminate the cylinder.

Venting


On the pump head there is a push-button (●) "air". This button is used for venting of the cylinder if the vacuum has become excessive.

How to operate the ACTIVE Erection System^{NT}


- 1) Place all parts of the **ACTIVE Erection System^{NT}** ready at hand.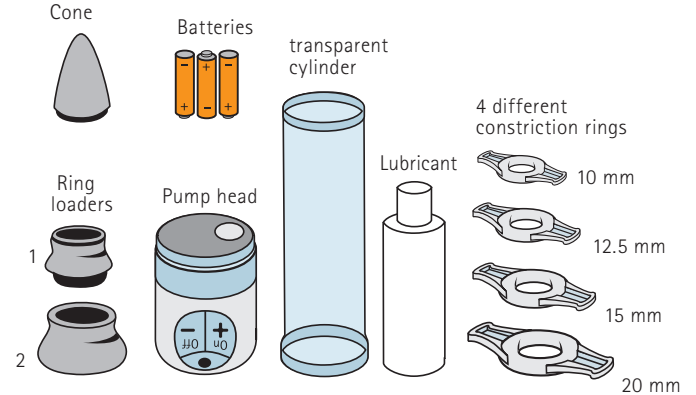


- 2) Before using for the first time, insert the batteries included (see also page 12 "Changing the batteries").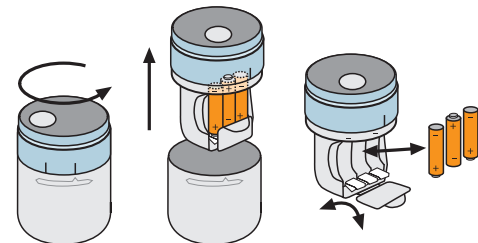


- 3) Attach the pump head to the end of the transparent cylinder, without gasket.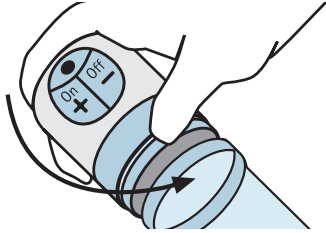


- 4) Now first attach the **ring loader 2** and then the smaller **ring loader 1** to the end of the cylinder with seal ring.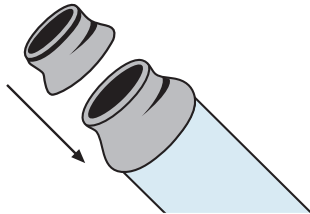

- 5) Finally place the end of the cone onto the small ring loader 1.

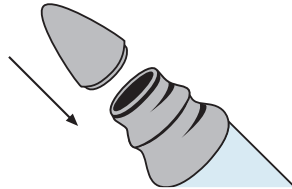
- 6) Start with constriction ring size 20, pushing it with some lubricant over the cone onto the **ring loader 1** or the larger **ring loader 2**.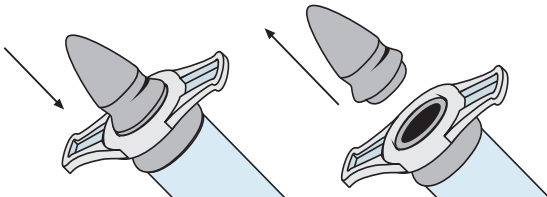


Now you can remove the **cone** again. If the ring loader 1 is too small for your penis, you can remove it and use only the ring loader 2.


- 7) Moisten the edge of the ring loader on the open side of the cylinder with some lubricant to allow the cylinder to seal the body. You may wish to remove some pubic hair in the area.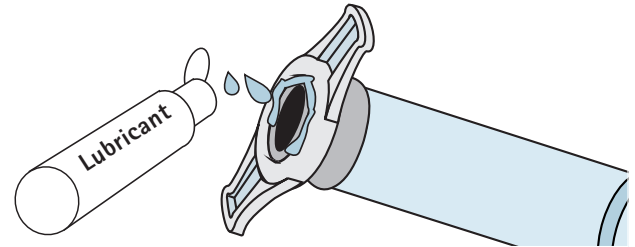

- 8) Slide the cylinder with the constriction ring slipped on at the open end over the non-erect penis, pressing cylinder and pump head against the body.

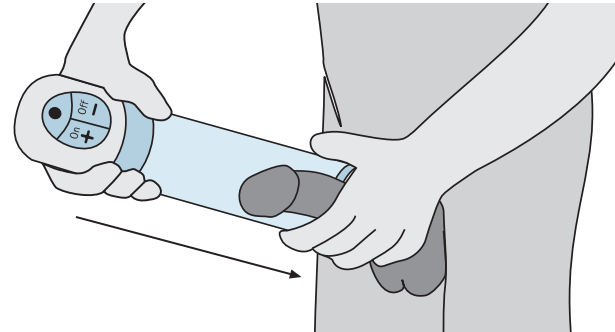
Now start pumping to create a vacuum inside the cylinder. Press the (+) "on" button on the pump head. Pressing the button again increases the pumping power. The erection now becomes visible in the cylinder.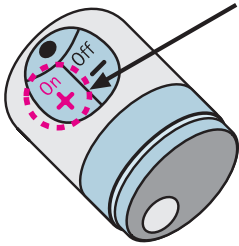

Switch on:
Plus (+) "on" button


In case of pain or excessive vacuum, you can vent the cylinder by using the (•) "air" button. Generally 30 to 60 seconds will be enough to obtain an erection.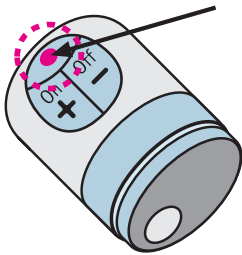


Air button:
Dot (•) "air" button
Keep pressed for 30 – 60 sec.

9) When the erection created and visible in the cylinder is sufficient, push the constriction ring slipped on onto the erect penis while the

pump motor is still active. Push the ring quickly as far as possible towards the penis root.

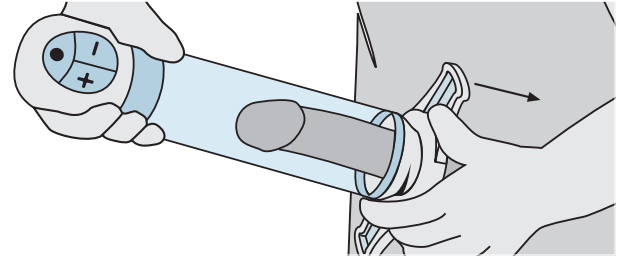
10) Turn off the device by pressing the (-) "off" button for about 3 seconds.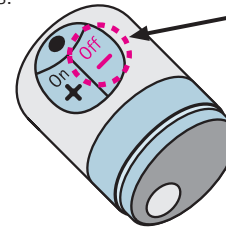


Switch off:
Minus (-) "off" button
Keep pressed for 3 sec.

11) The erection thus created and maintained is sufficient to experience satisfying sexual intercourse.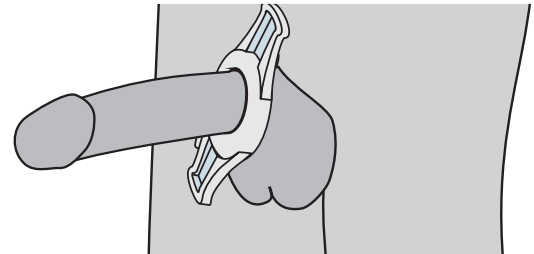

- 12) After orgasm (climax), remove the constriction ring again. To do this, first pull the two flaps on the constriction ring. The accumulated blood now flows from the penis back into the body. Only then remove the ring from the penis.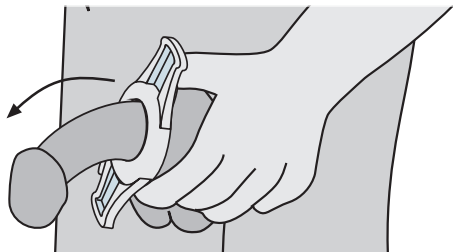

Important Notes

The constriction ring should not be used for more than 30 minutes. To prevent tissue damage caused by a prolonged interruption or reduction of blood flow, make sure you do not fall asleep with the constriction ring on the penis.

Use only water-soluble lubricants.

Upon orgasm you will have no visible ejaculation, since the constriction ring prevents discharge of the ejaculate (semen). After removal of the constriction ring, the ejaculate will flow out through the urethra. The temporary retention of the ejaculate does not cause any injury. We recommend that you use only the supplied constriction rings together with the **ACTIVE Erection System^{NT}**. The constriction rings are consumables and must be replaced from time to time. Do not use the product in case of damage; send it to us for inspection.

Side effects

If you suffer from cardiovascular disease, talk to your doctor prior to the first use of the device. The following side effects are possible: haematoma (bruising) on the shaft of the penis and / or on the glans and tissue injury in case of too rapid and / or excessive build-up of the vacuum. In this case, consult your doctor.

Contraindications

Do not use in case of sickle cell anaemia (a disease of the red blood cells), leukaemia (blood cancer), bone marrow tumours, diseases that result in altered blood coagulation and may cause bleeding or a prolonged erection or other medical contraindications for sexual activity.


Cleaning

With the exception of the motor / pump head, the **ACTIVE Erection System^{NT}** can be cleaned easily using warm water and mild soap. Detach the pump head from the device for this purpose. Clean the outside of the motor with a damp cloth and soap. Do not use any aggressive detergents. All parts can be kept discretely in the handy bag included in the delivery.

Changing the batteries

To insert the batteries, unscrew the pump head with the motor turned off (as shown below) and pull off the cover. Flap down the plate and change the batteries. Be sure to insert the new batteries in the direction indicated. Please always use only alkaline batteries. Please do not use rechargeable batteries, as their power is often not sufficient. When the batteries are empty, the pump motor will fail again and again (stutter), and the LED lights will flicker. In this case, please change the batteries.

Remove the batteries during extended periods without use.

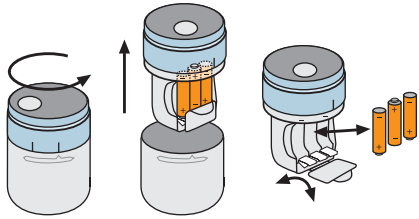Disposal

All parts, except the motor and batteries, can be disposed of with household waste. Motor and batteries must not be disposed of with household waste. **Dispose of the used batteries properly in accordance with the locally applicable rules and regulations.**

Warranty

The statutory warranty of 2 years applies to all non-wear parts. It expires in case of improper use, e.g. if the device is dropped. We recommend sending the device in to us for a technical inspection every 2-3 years.

Materials

- **Cone:** ABS plastic
- **Ring loaders 1 and 2:** ABS plastic
- **Cylinder:** Polycarbonate
- **Constriction rings:** Biocompatible polymer
- 3 × AAA Alkaline batteries 1.5 volts
- Insulated application part for use on the body

During application there is no exposure to electromagnetic radiation.

All components and wear parts (constriction rings) as well as the SYLK lubricant can be reordered. The current reorder list can be requested.

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KESSEL medintim GmbH